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






Cod. 27401/27402/27444 LETTINO VISITA MEDICA
Cod. 27401/27402/27444 EXAMINATION COUCH

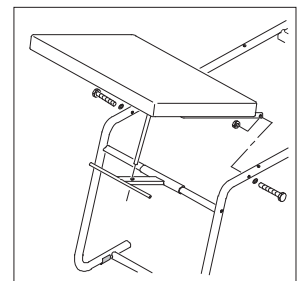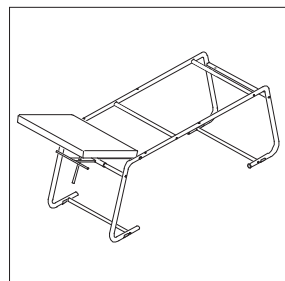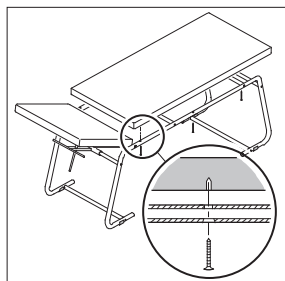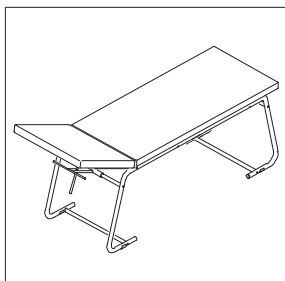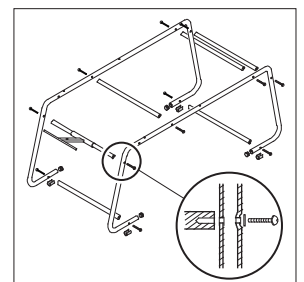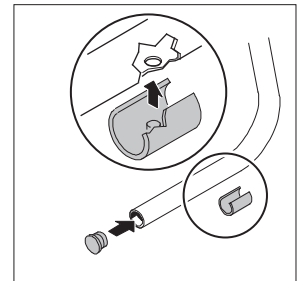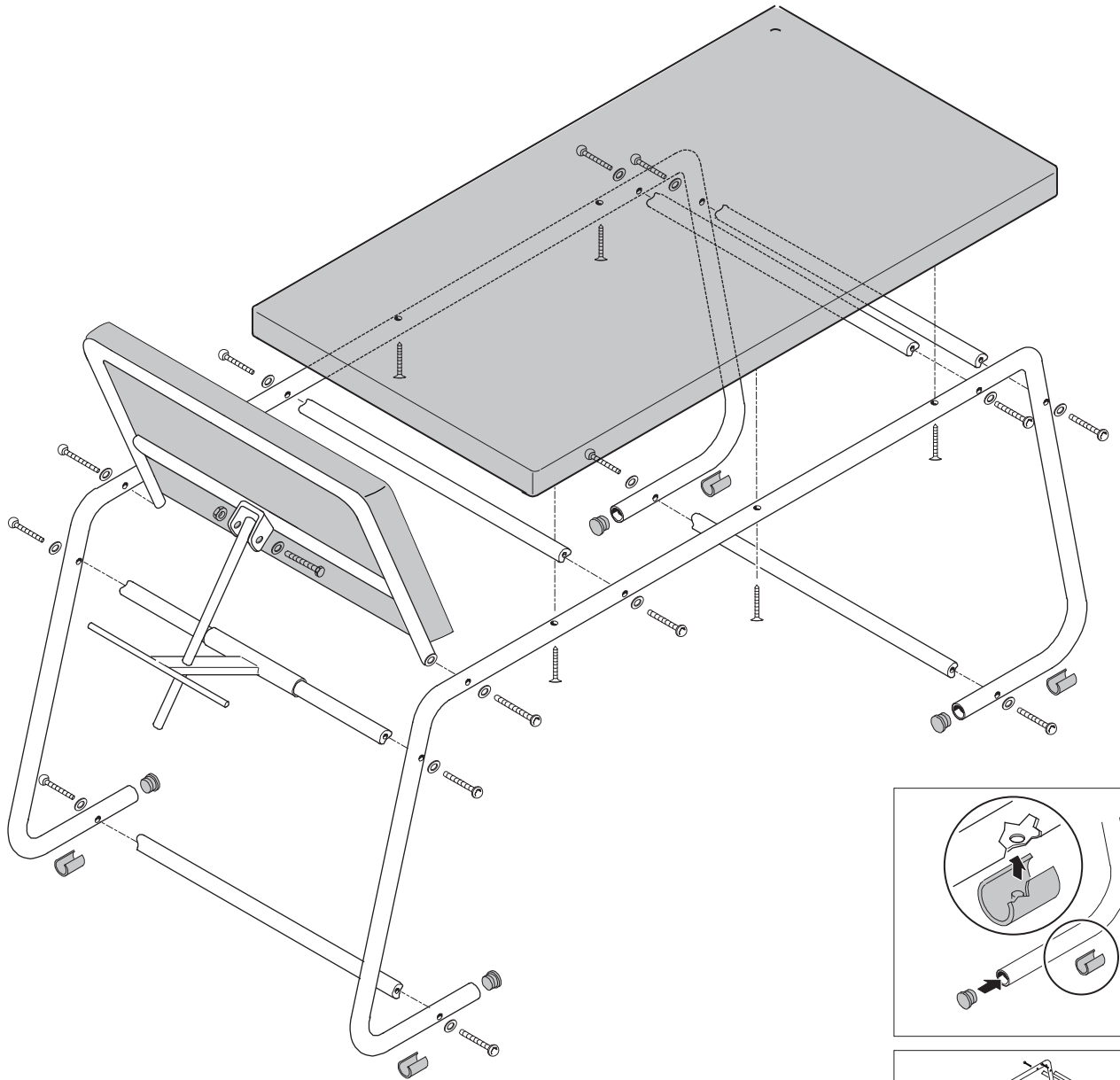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Cod 27401/27402/27444 - Examination couch

1. INTENDED USE

Examination couch complete with mattress. Upholstery in black washable imitation leather composed of two sections: one 130 x 61 x h. 5.5 cm fixed and 50 x 61 x h. 5.5 cm backrest adjustable through different positions. Delivered in assembly kit.

Always abide by safety standards and the instructions here contained. The manufacturer declines any liability for damage due to incorrect use of the equipment. The transport and unloading operations must be carried out by qualified and authorized personnel only. Other personnel must be kept away from the carriage.

2. USE RESTRICTIONS AND SPECIAL WARNINGS

Read the operating instructions

- The couch must **NOT** be used in the following conditions:
 - Fire risk, explosive or corrosive atmosphere.
 - Exposed to weathering.
 - Tampering attempts and/or modifications to the original product.
- Pay attention not to introduce body parts into moving and/or rotating components.
- The couch must be used under good stability conditions. The support feet must rest on a flat surface.

The lamps used for therapeutic irradiation can damage the upholstery imitation leather. Corrosive and/or abrasive detergents can damage the upholstery imitation leather.

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