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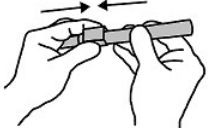
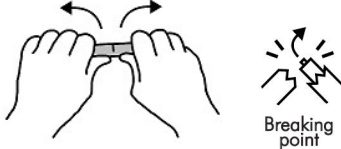
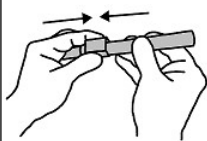
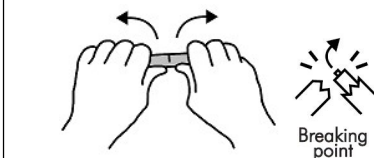
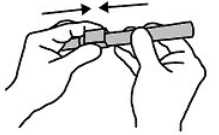
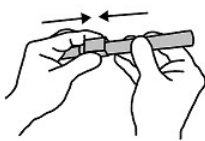
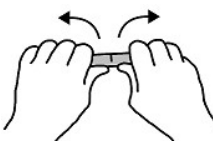
EN	SINGLE USE STERILE ELECTROCAUTERY		INSTRUCTION FOR USE	
INTENDED USE:		Cauterization of tissues and small blood vessels during surgical procedures.		
DESCRIPTION:		Electrocautery, sterile packaged disposable device powered by built-in alkaline batteries. By pressing the activation button, the device cauterizes by placing the incandescent tip on the patient.		
PATIENT POPULATION		No restriction.		
MODELS:		TYPOLOGY	TEMPERATURE	
		High Temperature Cautery - HTC	800-1200°C / 1450-2200°F	
		Low Temperature Cautery - LTC	600-700°C / 1100-1300°F	
USE MODE:		The devices must be used by qualified personnel. Open the package and place the cautery in a sterile field. After checking the integrity of the package, check the body of the cautery. In the presence of damage or visible defects, do not use the product and return it to FIAB. Remove the protective cap being careful not to damage the tip. Before use, check the function as follows: - press the activation key keeping the device away from people and things; - for HTC models check that the tip becomes incandescent; - For LTC models, touch a sterile gauze wet with water with the tip, verifying that 2-3 seconds after activation, evaporation of water from the gauze can be felt. During use on a patient, perform short (2-3 seconds) and repeated activations by pushing the switch. After use, always replace the protective cap.		
WARNINGS:		Product sterilized with Ethylene Oxide. Disposable product, DO NOT REUSE - Reuse may result in: - Cross infections; - Alterations of materials; - The loss of the initial functional characteristics of the product; Sterility is not guaranteed if the package has been opened and/or damaged. Avoid prolonged power-on, it can cause excessive heating and possible breakage of the tip. Do not exert pressure that could deform the tip and cause it to break. Avoid direct contact of liquids with the body of the electrocautery. Do not throw the device into fire. Warning: Contains batteries, do not throw into fire. Warning: Do not use in the presence of flammable substances or materials. Highest use temperature 25°C.		
CONTRAINDICATIONS:		Electrocauterics are not suitable for cauterizing heavy bleeding.		
CLINICAL BENEFITS:		- Cauterization without connecting the patient to an electric circuit. - Cauterization of patients carrying implantable devices sensitive to electromagnetic interferences.		
RESIDUAL RISKS:		The device can cause a fire if turned on in the presence of flammable substances.		
REGULATORY COMPLIANCE:		EN 60601-1: Internally powered equipment with type CF applied parts.		
LIFE TIME:		Number of activation x time of activation = 10 x 3 sec= 30 sec .		
STORAGE:		The product must be stored in its original package at the environmental conditions (temperature and relative humidity) specified on the label. Putting external heavy weights on the package could damage the product. Check the sterilization expiration date on the pouch.		
GENERAL NOTES:		If a serious incident occurs while using this device or as a result of its use, please report it to the Manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.		
WASTE DISPOSAL:		The product contains batteries, remove them as indicated, before disposing of the device. Waste coming from health facilities must be disposed of according to the regulation in force. The device may be contaminated with blood and must be disinfected before being discarded. In order to avoid accidental switching on (if the batteries have not been removed), cut the tip wire with pliers or scissors and always reinsert the protective cap.		
				
		Close the cautery with the stopper.	Grip the cautery as directed. Make sure that the part of the shaft to be broken is facing outwards. Apply a suitable pressure until the shaft breaks.	Remove the batteries taking care not be injured from sharp residues. Dispose of the batteries according to the regulations in force. Dispose of the remaining parts as contaminated waste.

CAUTION: U.S. FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN

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