


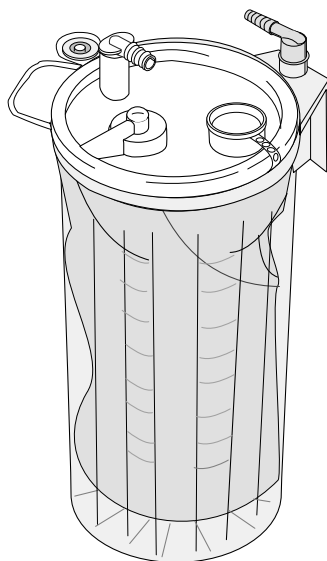
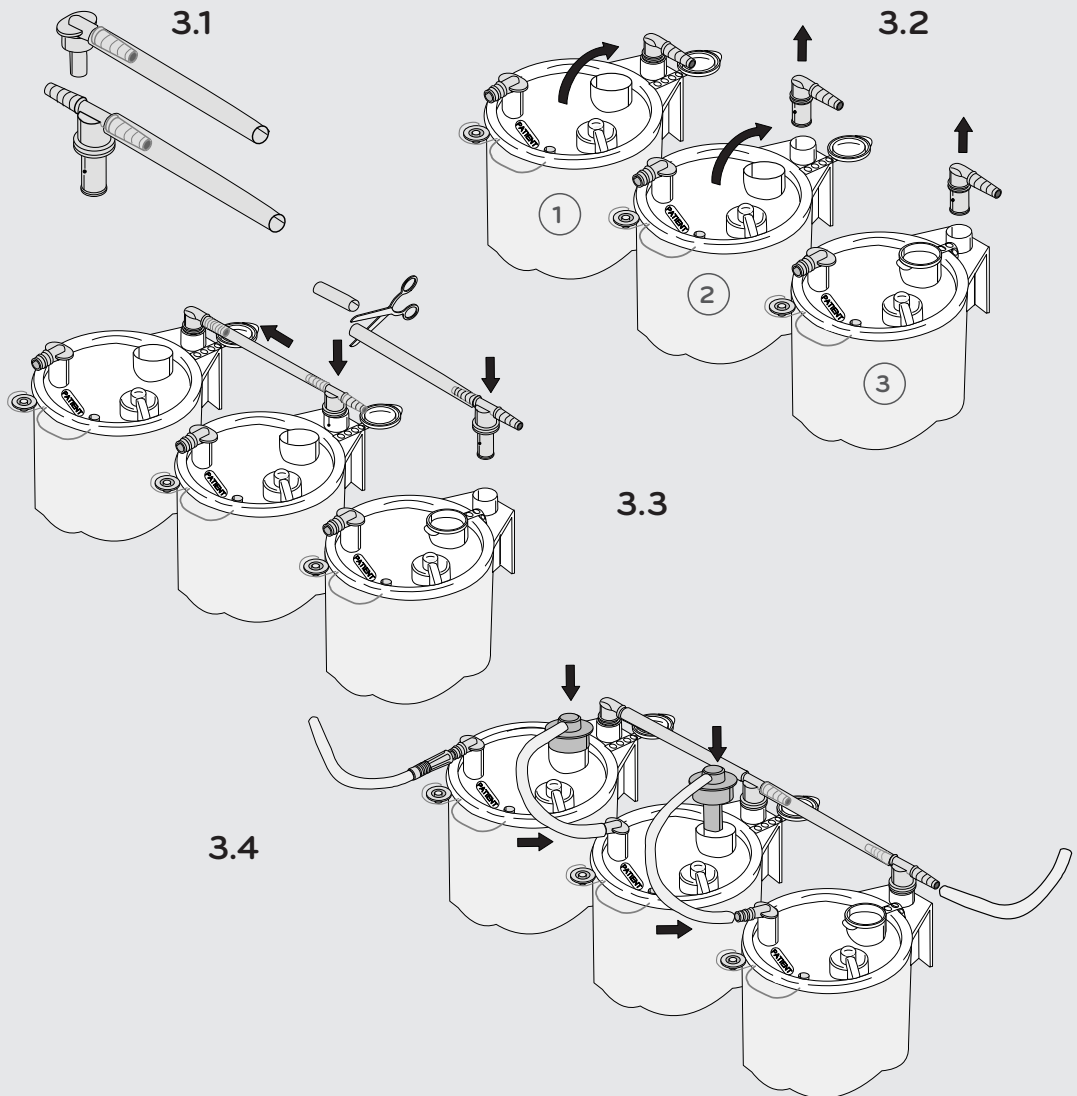
SERRES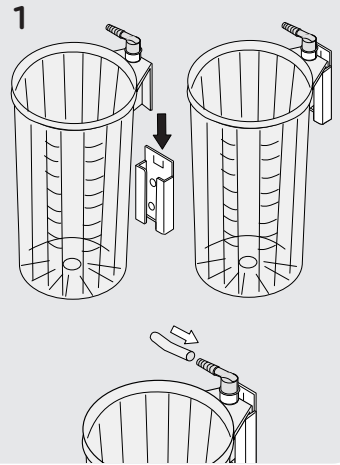


- EN** SERRES SUCTION BAG SYSTEM MANUAL
- DE** ANLEITUNG FÜR SERRES ABSAUGBEUTEL-SYSTEM
- FR** MANUEL DU SYSTÈME DE POCHE D'ASPIRATION SERRES
- ES** MANUAL DEL SISTEMA DE BOLSA DE ASPIRACIÓN SERRES
- IT** MANUALE DEL SISTEMA DI ASPIRAZIONE CON SACCHE DI RACCOLTA SERRES
- SV** BRUKSANVISNING SERRES SUGPÅSESYSTEM
- DA** BRUGERVEJLEDNING FOR SERRES SUGEPOSESYSTEM
- NO** VEILEDNING FOR SERRES SUGESYSTEM
- FI** SERRES IMUPUSSIJÄRJESTELMÄN KÄYTTÖOHJE
- NL** HANDLEIDING SERRES-OPVANGZAKSYSTEEM
- CS** PŘÍRUČKA PRO SYSTÉM ODSÁVACÍHO VAKU SERRES
- SK** NÁVOD K POUŽÍVANIU SYSTÉMU NASÁVACIEHO VAKU SERRES
- TR** SERRES ASPİRASYON TORBASI SİSTEMİ KILAVUZU
- HU** SERRES SZÍVÓZSÁKRENDSZER HASZNÁLATI ÚTMUTATÓ
- PL** INSTRUKCJA OBSŁUGI SYSTEMU WYKÓW DO ODSYSANIA SERRES
- PT** MANUAL DO SISTEMA DE ASPIRAÇÃO COM SACOS SERRES
- SL** NAVODILA ZA UPORABO ASPIRACIJSKEGA SISTEMA SERRES
- EL** ΕΓΧΕΙΡΙΔΙΟ ΣΥΣΤΗΜΑΤΟΣ ΣΑΚΟΥ ΑΝΑΡΡΟΦΗΣΗΣ SERRES
- CHI** SERRES 引流袋系统手册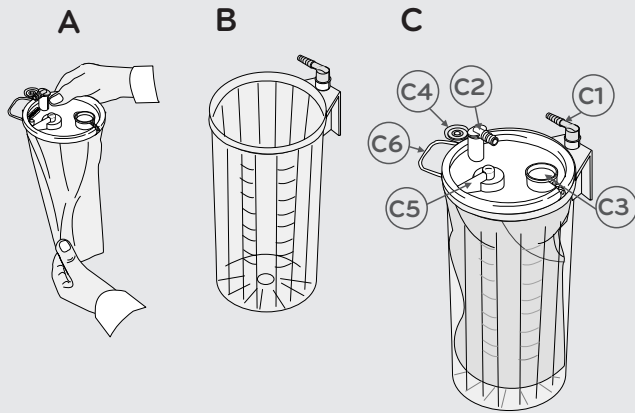


2.1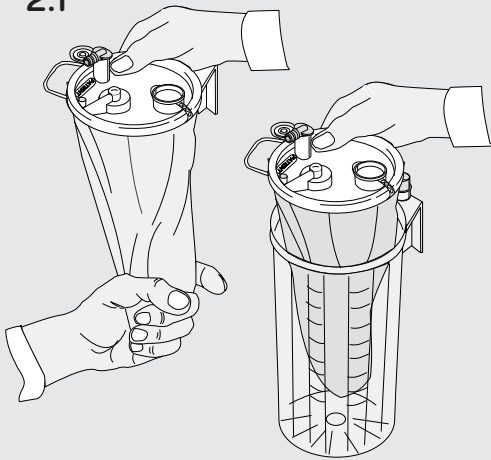


2.2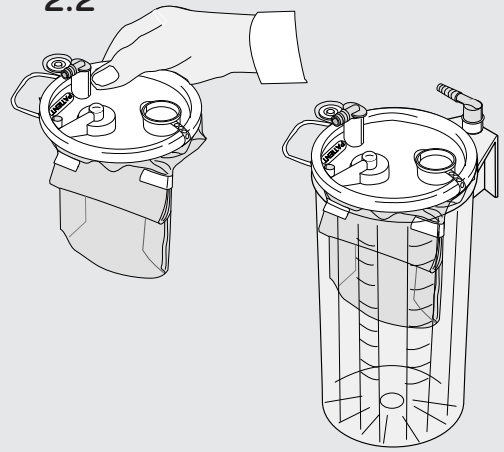


4

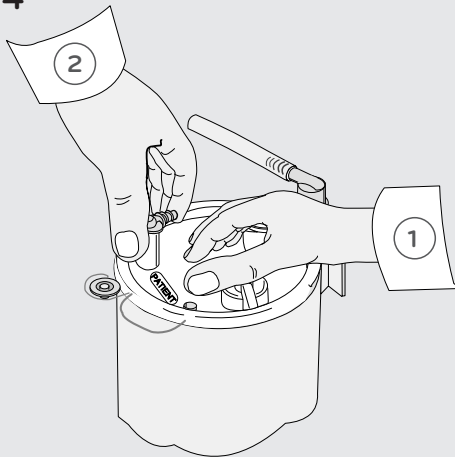
5

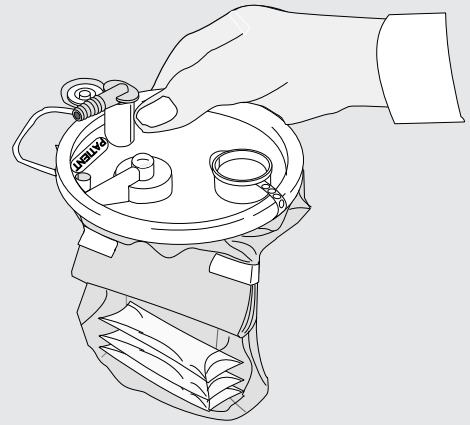6

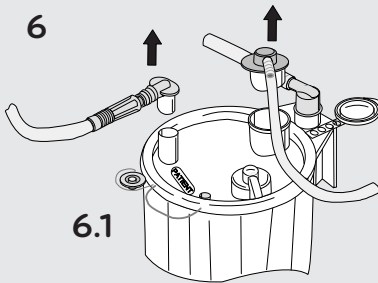
6.1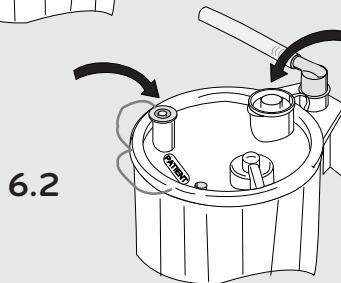

6.2

6.3

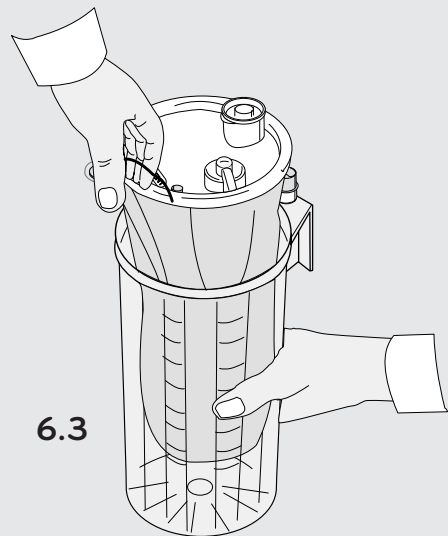The Serres suction bag system is used to collect liquids and secretions from patients during medical procedures. Please read this manual carefully before installing the product and beginning the procedure. The Serres suction bag system may only be installed and used by persons familiar with the product and used for the purpose stated by the manufacturer. Usage contrary to instructions is forbidden. The manufacturer cannot be held liable if the product is used in a manner and/or for purposes other than those described in these instructions. Before using the product, always ensure that all of the system's parts are intact. Use of faulty products is strictly forbidden.

Parts of the system

The basic parts of the system include the suction bag **Figure A** (disposable), the suction canister and the grey, right-angled connector of the canister **Figure B** (re-usable). The volume of the suction bags and canisters is either 1,000 ml, 2,000 ml or 3,000 ml, depending on the product model. Parts of the suction bag system have been introduced in more detail in **Figure C**.

C1 Angled connector (grey, attached to the suction canister)

C2 Patient connector (white, attached to the suction bag)

C3 Serial port (used when liners are connected in series)

C4 Patient connection plug

C5 Overflow protection

C6 Lifting handle

There are different versions of the suction bag and their installation can differ from each other. Ensure that you have read the instructions related to the correct version. Introduced in a separate manual entitled Accessories, there are a range of accessories available for the system.

NOTE! ONLY ORIGINAL SERRES PARTS AND ACCESSORIES MAY BE USED IN THE SUCTION BAG SYSTEM.

NOTE! IF YOU ARE USING A STERILE PRODUCT, ENSURE THAT ITS PACKAGE HAS NOT BEEN DAMAGED BEFORE USE. IF THE PACKAGE HAS BEEN DAMAGED, DO NOT USE THE PRODUCT.

NOTE! ALL SUCTION BAGS AND PRODUCTS THAT HAVE BEEN MARKED WITH THE  SYMBOL ARE DISPOSABLE AND RE-USING THEM IS STRICTLY FORBIDDEN.

Installation of the system

1. Installing the suction canister

Place the suction canister in a bracket in an upright position **Figure 1**. There are a range of brackets available for installing the suction canister (rail, wall, table and bed models). The suction canister can also be mounted to the canister holder in the movable Serres trolley. Connect the tubing from the vacuum source to the grey, angled connector at the rear of the canister.

2. Installing a single suction bag

2.1 Versions which have not been wrapped and taped

Unfold the suction bag and place it into the suction canister. **Figure 2.1**

2.2 Versions which have been wrapped and taped

Place the bag as it is in the suction canister **Figure 2.2** or follow the instructions in section 2.1.

NOTE! THE SUCTION BAG HAS TO BE INSTALLED INTO A SUCTION CANISTER OF EQUAL SIZE.

NOTE! ENSURE THAT THE SUCTION BAG FILM DOES NOT BECOME LODGED BETWEEN THE CANISTER AND THE LID.

3. Installing several suction bags (connection in series)

When large volumes of liquid are suctioned, Serres suction bags can be connected in series with serial tubes, vacuum tubes and T-connectors.

Figure 3.1.

- Place the bags to suction canisters as in section 2 and attach the bags to the canisters by pressing the middle of the lid.

- Remove the grey, angled connector (canisters 2 and 3) and open the suction bags' serial ports (bags 1 and 2) as shown in **Figure 3.2**.

- Connect the suction canisters to each other with separate T-connectors and tubes in accordance with **Figure 3.3**. Use scissors to cut the tube to a suitable length.

NOTE! THE T-CONNECTOR AND VACUUM TUBE ARE RE-USABLE AND DO NOT NEED TO BE REPLACED BETWEEN OPERATIONS.

- Connect the serial tubes carefully through the opened serial port opening in the suction bag to the next suction bag's patient connector. **Figure 3.4**

For best performance, you are recommended to use a vacuum shift for the serial connections of the largest amounts of liquid.

4. Unfolding the suction bag

Suction bag is installed using vacuum **Figure 4**. Turn the vacuum source on and, at the same time, press the middle of the lid lightly (part 1). After the suction bag has straightened, close the patient connector using your finger (part 2) so that the lid tightens to the suction bag.

NOTE! BEFORE USE, ENSURE THAT THE VACUUM HAS BEEN CREATED AND THE SUCTION BAG IS FULLY INFLATED.

Connect the patient tube to the patient connector.

The system is ready for use.

Using the suction bag system

When the suction is on, the liquid flows inside the suction bag. The suction bag can be used throughout the procedure or until the suction bag is full. When the suction bag is full, overflow protection cuts the suction and prevents the flow of liquid to the suction source. The scale on the suction canister displays the volume of the suctioned liquid. The accuracy of the scale is +/- 100ml.

Use of suction bags equipped with solidifying agent (Pre-gelled)

Some suction bag versions include a solidifying agent **Figure 5** that solidifies the suctioned liquid. Use pre-gelled suction bags similar to regular suction bags. When measuring the volume of the suctioned liquid, take note of the fact that the scale on the suction canister includes the amount of the solidifying agent contained in the suction bag.

1l pre-gelled suction bag: 25 ml

2l pre-gelled suction bag: 50 ml

3l pre-gelled suction bag: 85 ml

In addition, note the normal tolerance of the suction canister scale. If you use separate sachets or a solidifying agent sold as separate powder, use 25g / 35ml of substance per one litre of liquid. Note the warnings in the solidifying agent packet.

NOTE! IF THE PRE-GELLED SUCTION BAGS ARE USED IN SERIAL CONNECTIONS, THE VACUUM SOURCE MUST BE ON FOR THE DURATION OF THE WHOLE PROCEDURE.

After the procedure

After the suctioning procedure, disconnect the patient tube and the angled connector (or alternatively straight connector or twin connector) and serial tube, if used **Figure 6.1**. Close the connection with the patient connector plug provided on the lid of the suction bag. In serial connections, also plug the serial port **Figure 6.2**. Finally, turn off the vacuum source. Lift the suction bag from the handle **Figure 6.3**.

NOTE! DO NOT TURN OFF THE VACUUM SOURCE BEFORE YOU HAVE CLOSED THE SUCTION BAG.

Disposal of used products

Serres suction bags, specimen sets, measuring cups and serial tubes are disposable and must be replaced patient-specifically. If the same patient undergoes long-term treatment, the suction bag must be replaced at least every 24 hours. Reusable products must be decommissioned if they have become damaged or do not function as planned. Used or decommissioned products must be disposed of in accordance with recycling labelling and hospital-specific instructions.

NOTE! REUSE OF DISPOSABLE PRODUCTS IS STRICTLY FORBIDDEN. REUSE REDUCES THE PERFORMANCE OF THE PRODUCT AND CAN CAUSE AN INFECTION RISK. WHEN HANDLING A USED SUCTION BAG, ATTENTION SHOULD BE PAID TO THE FACT THAT IT MAY CONTAIN INFECTIOUS WASTE.

Cleaning of re-usable products

The suction canister and angle connector can be washed (95°C) and autoclaved (121 °C). Remove the grey, angled connector before washing. Other reusable products can be wiped with disinfectant.

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请注意！如果您使用的是灭菌产品，请确保该产品的包装在使用前未被破坏。如果包装已经破损，请勿使用该产品。请注意！所有标有 © 符号的引流袋及产品均为一次性使用，严禁重复使用。