# MT BASE PLUS

**GIMA SPA**

Via Marconi 1 - 20060 Gessate (MI) - ITALY

Tel +39 02 9538541 Fax +39 02 95381167

[www.gimaitaly.com](http://www.gimaitaly.com)

[gima@gimaitaly.com](mailto:gima@gimaitaly.com) - [export@gimaitaly.com](mailto:export@gimaitaly.com)

MA321IGBa

---

## IMPORTANTE/ IMPORTANT

---

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*No part of this document could be photocopied, reproduced or translated in other language without the written consent of LED S.p.A.*

*In case of necessity of technical assistance contact your own retailer.*

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<b>GENERAL INFORMATION</b>	<b>29</b>
<b>INTENDED USE</b>	<b>30</b>
<b>MAGNETOTHERAPY</b>	<b>31</b>
<b>MAGNETOTHERAPY AND DISEASES</b>	<b>33</b>
<i>MAGNETOTHERAPY AND SPORT</i>	33
<i>MAGNETOTHERAPY AND REFLEX TREATMENT</i>	33
<i>MAGNETOTHERAPY AND FOOT'S PATHOLOGY</i>	33
<i>MAGNETOTHERAPY AND FRACTURES</i>	33
<i>MAGNETOTHERAPY APPLIED AT THE CERVICAL</i>	33
<i>MAGNETOTHERAPY AND KNEE'S PATHOLOGIES</i>	34
<i>MAGNETOTHERAPY AND GONOARTHROSIS</i>	34
<i>MAGNETOTHERAPY AND INSOMNIA</i>	34
<i>MAGNETOTHERAPY AND PAIN</i>	34
<i>MAGNETOTERAPIA AND LUMBALGIA</i>	34
<b>CONTRAINDICATIONS AND COLLATERAL EFFECTS</b>	<b>35</b>
<b>DESCRIPTION OF THE UNIT</b>	<b>36</b>
<b>CONTROLS</b>	36
<i>Main socket (1)</i>	36
<i>Main mechanical switch (2)</i>	36
<i>Display Frequency Freq (3)</i>	36
<i>Display Intensity Level (4)</i>	37
<i>Display Time Timer (5)</i>	37
<i>Connector OUT 1 (6)</i>	37
<i>Connector OUT 2 (7)</i>	37
<i>Magnetic Field Sensor</i>	38
<b>TECHNICAL CHARACTERISTICS</b>	38
<b>DIMENSIONS</b>	38
<b>STANDARD COMPOSITION</b>	38
<b>INSTALLATION AND TREATMENT START</b>	<b>39</b>
<i>PLACEMENT OF THE TRANSDUCERS</i>	40
<b>APPLICATIONS</b>	<b>41</b>
<i>INDICATIVE TABLE OF THE APPLICATIONS</i>	41
<b>MAINTENANCE</b>	<b>43</b>
<b>PREVENTIVE MAINTENANCE</b>	43
<i>CLEANING OF THE UNIT</i>	43
<i>CLEANING OF THE TRANSDUCERS</i>	43
<i>CLEANING OF THE BANDS</i>	43
<b>CORRECTIVE MAINTENANCE</b>	43
<b>PROBLEMS SOLUTION</b>	44
<b>F.A.Q.</b>	<b>45</b>

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 **WARNINGS**

---

It is very important that this manual of instructions is preserved together the unit for any future consultation. If the unit must be sold or transferred to other consumer, make sure that the manual comes furnished together, so that the new consumer could be aware of the operation of the unit and of the relative instructions.

**THESE WARNINGS HAVE BEEN WRITTEN FOR YOUR SAFETY AND FOR THE OTHERS ONE, SO PLEASE PAY ATTENTION READING THEM BEFORE YOU INSTALL AND USE THE UNIT.**

- After have removed the packing we make sure of the equipment's integrity. If there is a doubt do not use it and turn to qualified staff.
- The electric security of the equipment is assured only when the same is correctly connected to an electric installation conformable to the standard norms. It is necessary to verify this fundamental requisite of security and, if there is a doubt, ask for a control by qualified staff.
- Before connecting the equipment be sure that the rating (on the back panel) is in conformity to the electric system.
- If there is incompatibility between the wall socket and the supply cable of the equipment substitute it with another one. In general it is not advised the use of adapters, multiple wall sockets and/or extensions. If their use is essential it is necessary to use only simple adapters and extensions in conformity to the norms of safe.
- The use of any type of electric equipment requires the observance of some fundamental rules. In particular:
  - do not touch the equipment with wet hands or feet;
  - do not use the equipment with barefoot.
- Do not expose the equipment to atmospheric agents (rain, sun, etc.)
- Do not keep the equipment uselessly connected. Switch off the general interrupter when the equipment is not used.
- This equipment is destined only to the use for which it has been designed. Every other use is improper and so dangerous. The manufacturer cannot be considered responsible for possible damages caused by improper or unreasonable uses.
- It is dangerous to modify the characteristics of this equipment.

 Before carrying out any type of cleaning or maintenance, disconnect the equipment through the plug or switching off the mains switch.

- If there is a damage or the equipment doesn't work well, switch it off. For the repairing turn only to a qualified center of Technical Assistance and ask for the original spare parts. The missed respect of these things can be dangerous for the safety of the equipment and for the user.
- It is recommended to carry out the treatments after a medical consultation
- Il dispositivo emette radiazioni di tipo elettromagnetico e durante il suo impiego potrebbe interferire con indagini o trattamenti specifici.

- This equipment was manufactured to satisfy the present electromagnetic compatibility. In case of the functioning of the unit is interfered or interfies with the normal functioning of other electrical and electronic equipments, it is recommended to feed the equipment by a different electrical plug and/or try to dispose the unit in a different way until to the interference ends.
- Do not use mobile phone near to the equipment to avoid possible interference with it.
- It is recommended to not carry out this therapy at the same time of TENS and/or Holter.
- Many components used inside this equipment are sensible to the electrostatic charges. If you need to hand any electronic component, avoid the eventual static energy that you can carry, by touching a mass plane to charge at hearth. If it is possible, wear a bracelet connected to earth. If you do not observe these precautions you could cause permanent damages to the electronic components of the unit.
- When the unit is working do not put the magnets near any type of screens (monitor, tv, etc..), this could cause the images and colours distortion.
- Before the use of the unit verify the conditions of cables and equipment to individualize electric dangers.
- The unit does not fit explosive or gas environments.

---

## GENERAL INFORMATION

---

**MT BASE PLUS** is an ELF magnetotherapy unit that uses, for therapeutical intent, the effects induced to the human body by low frequency magnetical fields.

Unit has inside a generator of square waveform electrical current with frequency varying from 10 to 99 Hz and impulse time varying so to obtain a field's intensity determined from 10 to 80 Gauss. Magnetical field's intensity is obtained by changing realization's time of the circuit supplying the current into the transducer.

Quick variation of field induces into the patient the electromotive forces producing the stimulating currents.

Current is monitored by a monitoring circuit managed by an electronic microcontroller that guarantees the precision.

Unit has inside a sound sensor of magnetic field and a safety system that stops the delivery in the case of short-circuit.

Two level transducers can be connected, in the same time, to the **MT BASE PLUS** ( the field produced by each transducer is the half of the total one), they were realized by semirigid material to give more comfort to the application in contact with the body.

Buttonholes for the fixing by elastic bands are on the transducers and the two polarities North (N) and South (S) are shown. This allows to use them singly or by coherent concatenate or repelling magnetic field.

---

## INTENDED USE

---

ELF magnetotherapy unit **MT BASE PLUS** is intended for a temporary use on man, it was designed and manufactured for domiciliary applications by final user, after physiotherapist or paramedical staff control

The most important effects of the magnetic fields are:

- Antiedemigen
- Analgesic
- Antiphlogistic
- Regenerative
- 

**MT BASE PLUS** must be used by equipped or spare accessories recommended by the producer.

By equipped accessories, the and have applications in:

- **Traumatology:** Pseudoarthrosis and delay in the fractures' consolidation.  
Fractures that have a risk of pseudoarthrosis  
Lesions of the soft parts (contusions, distortions, burnings)  
District algodystrophy
- **Rheumatology:** Degenerative rheumatisms  
Inflammatory rheumatisms  
Fibromyalgic rheumatisms  
Bony aseptic necrosis
- **Angiology:** Peripheral arteriopathies  
Wounds and ulcers
- **Neurology:** Muscular-tensive headache  
Posttherpetic neuralgia
- **Dermatology:** Psoriasis  
Ittiosis
- **Endocrinology:** Senile and post menopause osteoporosis

---

# MAGNETOTHERAPY

---

Magnetotherapy is a type of physiotherapy that uses electromagnetic energy.

This is advisable if it is necessary to stimulate the turn-over of the tissues after injurious events of any kind:

- bony fractures'healing
- treatment of vascular cutaneous ulcers (in diabetic patients)
- anatomic and functional recovery of injured nerves
- reduction of disfiguring cutaneous cicatrix
- treatment of contusions, articular distorsions and slight muscular-articular trauma

By using magnetic fields on the body the difences are potentiated with electrical re-polarization of the cells, antistress action is produced, the cells are turned-over improving enzymatic kinectics and cellular membranes are repolarized.

The action of the magnetic fields on free radicals, oligoelements and ferromagnetic substances of the intracellular surrounding, is very important for the omeostatic balance of the cell.

So Magnetotherapy induces the acceleration of all repairing phenomena by strong bioregenerative, antiphlogistic, antiedematous, antalgic action.

Magnetic fields application influences the several processes regulating the vital functions of the cellular membrane and have effects on level of neuro-vegetative and metabolic regulation, recharging and regenerating the cells without vital force.

Magnetic fields are divided in:

constant magnetic field

magnetic fields varying in the time or called pulsed magnetic fields.

Constant magnetic fields are produced by permanent magnets or by electromagnets crossed by constant continuous current. Permanent magnets are made by particular high permeability ferromagnetic league, they are small disks, like coins, that are applied in contact with the skin for a period of time. This method is advisable on muscular-skeleton pains, both inflammatory and traumatic kind, by placing the magnetes on the painful zones and on the Trigger Points, that is small cutaneous zones that, if stimulated, produce pain from distance. Magnetic field produced is very high, from 3000 to 10.000 Gauss or more, but as it decreases by taking away from these points in directly proportional way to the square of the distance, it is needed to apply the magnet nearer to the treated zone.

Variable or pulsed magnetic fields are divided in high (over the MHz) and low frequency (from 1 to 3000 Hz) electromagnetic fields.

High frequency pulsed magnetic fields use frequencies in the range of 1-300 MHz, used in the marconitherapy too. This last therapy wants to obtain a strong thermal effect by using high powers, while in Magnetotherapy a MHz frequency and its harmonics, pulsed from 5 to 600

times at second, is maintained at few watt power , so to not produce any important increase of temperature in the treated tissues. It is used a generator connected to transducers that are applied in direct contact with the skin or near it, because the intensity of produced field ends in a proportional way to the square of the distance.

Magnetotherapy by low frequency pulsed magnetic fields is consider a valid physic therapy, with positive biological effects, above all indicates in the treatment of bony fractures and inflammatory and chronic degenerative diseases.

The many benefits and the total penetration into the tissues, with consequent deep effectiveness, the absence of injurious effects for low and slight intensities, allow a great use of Magnetotherapy by ELF (Extremely Low Frequency) fields that act for effect of induced microcurrents.

Magnetotherapy (ELF) is characterized by:

- Low frequency
- Slight electrical field
- Low irradiated power
- Slight thermal effect
- High penetration into the tissues.

About general therapeutical parameters, usually low frequencies and higher intensities are prescribed to obtain a biostimulating effect, while high frequencies and lower intensities are preferred to obtain analgesic and/or sedative effect. Middle frequencies and intensities are used to stimulate arterial circulation.

Application's time change according to the pathology from 10 minutes, for arthro- rheumatic diseases, to several hours, for pseudo arthrosis, and the session's number changes from some days for acute phlogistic processes to some months for pathology like pseudoarthrosis.

---

## MAGNETOTHERAPY AND DISEASES

---

Magnetotherapy was legitimated by several scientific studies as solution against many pathologies regarding muscular-skeleton apparatus.

We report, in the following paragraphs, the most common pathologies treated by low frequency pulsed magnetic fields.

### ***MAGNETOTHERAPY AND SPORT***

Magnetotherapy is a valid therapy of the sport pathology, both local applications after luxations, fractures, contusions, distorsions, and for trauma of the muscular tissue due motory stress or fatigue and overloads. In these cases Magnetotherapy acts by reflected action, improving the time of resorption of the lactic acid that otherwise could be almost irrecoverable by the body due to the persist of the fatigue's sensation, until to cause muscular stretchings and tears.

### ***MAGNETOTHERAPY AND REFLEX TREATMENT***

From the union between Magnetotherapy (massage and passive gymnastics) and reflex treatment it is possible to obtain strong effects of re-balance of weakness condition or organic overload, relax, relief from fatigue or, if it is necessary, tone and stimulation.

### ***MAGNETOTHERAPY AND FOOT'S PATHOLOGY***

Magnetotherapy is a valid antiphlogistic therapy and so pain-relieving, for the distorsions of the ankle and per le distorsioni della caviglia e la talagia (or tallonitis).

### ***MAGNETOTHERAPY AND FRACTURES***

The interruption of the bony continuity could be caused by traumatic stress, cysts, tumor, osteoporosis. These pathologic fractures are distinguished in simple fractures and exposed fractures (with damages to the soft parts: muscles, covering tissues, etc.) and, according to their seat, in epiphysis fractures (on the proximal segment of the bone's head), diaphysis fractures (on the bony surface), metaphysis fractures (in the center of the bone). According to the extension of the lesion, the fractures are distinguished in longitudinal, trasversal, spiriod, fragmentary. The symptomatology includes, according to the importance of the traumatic event, diffuse pain, visual difference, reduced mobility, pelt noise, ecchymosis, tumefaction, obliged mobility with functional impotence. Magnetotherapy acts on the bony fractures inducing piezoelectrical effect on the structures interrupted by the bony discontinuity, promoting the calcium production with consequent improvement of the bony corn, it acts also on the post-traumatic shock (absence of hematic perfusion, oxygen deficit) of the vital organs.

### ***MAGNETOTHERAPY APPLIED AT THE CERVICAL***

Magnetotherapy is advisable against wryneck (lateral applications, under the nape in correspondence of the pain), comprensive post-traumatic radiculopathy and wearing out pathologies (application on the nep and on the point of maximum pain). Far or reflexed applications have catecolaminico intent (antiphlogistic) and endorphinic (relief from pain) and the symptomatic point is treated indirectly.

## ***MAGNETOTHERAPY AND KNEE'S PATHOLOGIES***

Magnetotherapy is recommended against distortions and other knee's pathologies, thanks to its anti-phlogistic action and relief from pain. Low frequency fields therapy can be unit to the instrumental physiotherapy or in presence of orthosis or advisable after a surgical operation at the ligaments. In this last case it is needed to restore, by magnetotherapy and physiotherapy, the circulation function, reduce the edema, begin the ossification process screw/cockroach (Kenneth – Jones' reconstructive technique that consists in the placement of a bony-tendineous taking, from rotuleous tendines, in the place of the old frontal crossed ligament) pain reduction, restoring of the articular elasticity, promoting cellular exchanges.

## ***MAGNETOTHERAPY AND GONOARTHROSIS***

ELF applications (at least 45 minutes, to obtain relax and decontracting action of lumbar muscles, overtired by unstable movement) gives, also in this case, helps against pain symptoms and, in old patients, represents a strong help against the degenerative condition. magnetotherapy is effective in patients with prosthesis (transducer under the knee), thanks to its anti-phlogistic, pain relief and bioenergetic action.

## ***MAGNETOTHERAPY AND INSOMNIA***

Insomnia is a frequent disease due to organic or emotional problem. The organic problems could be caused by digestive difficulties, with consequent gastric iper-production, diseases from stress. There are also diseases, due to the age where the sleep diminishes and becomes fragmentary. A lot of responsibilities of this disease is psycho-pathologic origin. Magnetotherapy applications during the sleep could be useful for the patient affected by insomnia. The most important effect of magnetism in the depressive disorders, is the restoration of the comunicability between central nervous system and organic activities connected to the metabolism, with consequent reduction of the reactivity and peripheral excitability.

## ***MAGNETOTHERAPY AND PAIN***

Pain is the final product of a mechanical and inflammatory event. At tissue level the pain interests zones mostly great and this causes enzymatic and difensive reactions by the organism. The intent of the magnetotherapy is, in this case, the transfer of energy with pain relief, but also with pain relief due to reflexed action, promoting the action of the used structure. The frequencies have to be low, and in "loco", to have pain relief. Magnetotherapy can promote the assimilation of anti-inflammatory and pain relief products and in the acute phase antibiotics too, accelerating the cellular exchanges reaction; it helps the permeability and the action of emunctory organs, in the elimination of the organic discharging substances after extended pharmacological therapy, by high frequencies.

## ***MAGNETOTERAPIA AND LUMBALGIA***

Lumbalgia means local pains at the inferior part of the back. The pain turns up suddenly, after an abrupt movement of the vertebral column, or it could be gradually and have several causes, from the pressure on a nerve by a "hernia" or arthritis. The symptomatology is represented from pain in lumbar, lumbarsacral seat and from defending muscular contractures of interested segment.

In these clinical cases Magnetotherapy is useful, generalized at all vertebral system, by using big plates, and by a frequency range for the anti-phlogistic effect of the root and nucleus. The pain relief is consequent for the direct action on the enzymatic catecolaminic (adrenal) and endorfinic (ipophisis) productions. This method gives a good clinical-symptomatic aid in important forms of radiculopathies, in collaboration with medical and anti-phlogistic therapy.

---

## CONTRAINDICATIONS AND COLLATERAL EFFECTS

---

It is recommended to use **MT BASE PLUS** after medical consultation.

The Magnetotherapy applications have to be avoided in patients:

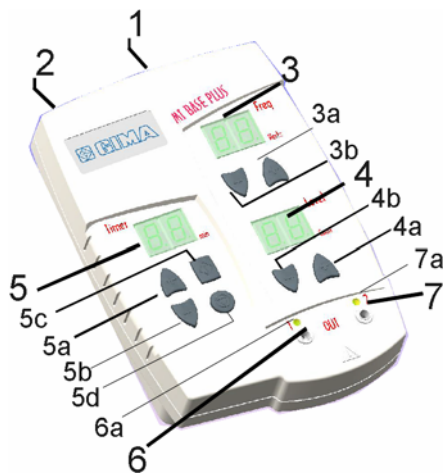
- with implanted pace-maker or other electronic system;
- with heavy unbalance of arterial pressure;
- heart patient;
- with heavy diseases of the nervous system;
- with heavy renal insufficiency;
- with neoplasia;
- with youthful diabetes;
- in infant-youthful age;
- in pregnancy woman;
- hemorrhagic;
- epileptic;
- with metal prosthesis and/or magnetizing means of synthesis;
- with acoustic prosthesis;
- with copper spiral.

Treatments by magnetic fields could produce:

- a transitory relapse of algic syndrome in patient with rheumatism
- vertigo and insomnia sensations in applications that interest the rostral segment
- ureteral colics in the applications on the lumbar segment

In this case it is recommended to interrupt the treatments and consult the physician.

## DESCRIPTION OF THE UNIT

1	Main socket
2	Main mechanical switch
3	Display frequency "Freq" (Hz)
3a	+ key to increase
3b	- key to decrease
4	Display intensity "Level" (Gauss)
4a	+ key to increase
4b	- key to decrease
5	Display Time "Timer" (min)
5a	+ key to increase
5b	- key to decrease
5c	Output start key (Start)
5d	Output stop key (Stop)
6	Connector OUT 1
6a	Warning light output 1
7	Connector OUT 2
7a	Warning light output 2

	Output start key (Start)
	Output stop key (Stop)

### CONTROLS

#### MAIN SOCKET (1)

On the back panel of the unit there is the main socket. Put in this socket the supplied main cable. The cable must be connected to a main socket with compatible characteristics with the unit (see the plate's data on the underside part of the unit).

#### MAIN MECHANICAL SWITCH (2)

At the left of the unit there is the main switch. Putting the switch on I position, the unit switches on. The switching on of the unit is shown by the lighting of the display.

#### DISPLAY FREQUENCY FREQ (3)

On top at right of the unit there is a display for the visualization of the frequency, in Hertz. Switching on the unit, the frequency is at 75 Hz and can be changed in the steps 10 - 25 - 50 - 36

75 - 99 Hz by the suitable keys. The + key is used to increase the frequency value, while the – to decrease it.

#### ***DISPLAY INTENSITY LEVEL (4)***

On the central part of the unit, at the right, there is a display to visualize the intensity values, in Gauss. Switching on the unit, the intensity is at 30 Gauss and can be changed (from 10 to 80 Gauss, with 5 steps) by the suitable keys. The + key is used to increase the intensity value, while the – to decrease it.

#### ***DISPLAY TIME TIMER (5)***

At the left of the unit, there is a display to visualize the time values, in minutes. Switching on the unit, the time is at 30 minutes and can be changed (from 0 to 60 min) by the suitable keys, when the apparatus does not supply. The + key is used to increase the frequency value, while the – to decrease it. By the start key (5c) it is possible to start the time count and start the output too. The output start condition is indicated by the lighting of a light point at right of minutes' display. By the stop key (5d), it is possible to stop the time count and stop the output too. The stop condition is indicated by the switching off of the point at the right of the minutes' display. The end of the treatment time is indicated by an acoustic signal of 10 seconds. By setting the 00 time it's possible to realize an infinite treatment time, without programmed interruption.

#### ***CONNECTOR OUT 1 (6)***

Connection point of the supplied magnetic transducer. If there is current output , the corresponding warning light, warning light 1, is lighted (patient connected).

#### ***CONNECTOR OUT 2 (7)***

Connection point of the supplied magnetic transducer. If there is current output , the corresponding warning light, warning light 2, is lighted (patient connected).

**MAGNETIC FIELD SENSOR**

The unit has inside a system of magnetic field survey.

To take the field (indicated by an acoustic signal) it needs that:

- 1 the unit must be in START condition.
- 2 the FREQ section must be set at 99Hz.
- 3 the LEVEL section must be set at 80 Gauss.
- 4 the transducer must be placed as shown in the figure.

**TECHNICAL CHARACTERISTICS**

Power supply	230V~ / 50Hz
Max input power	22VA
Inside fuses	1xT500mA (delayed) type 5x20
Max intensity	80 Gauss (8mT)
Output frequency	10 → 99Hz
Safety class	II BF
Protection	Unit IP20          Transducers IP20

**TRANSDUCERS:**

Delivered radiation:	electromagnetic field
Max. electromagnetic field intensity:	100 Gauss (10mT)
Frequency:	2 → 100 Hz

**DIMENSIONS****UNIT:**

Max height	68mm	Width	138mm
Length	193mm	Weight	1.040 Kg

**TRANSDUCERS:**

Max height	33mm	Width	97mm
Length	155mm	Weight	0.6 Kg

**STANDARD COMPOSITION**

1 REF	80200.10A	<b>MT BASE PLUS</b> unit
1 REF	00100.02	Main cable 2mt
2 REF	80100.80	Transducers 80 Gauss 155x97
1 REF	00602.060	Velcro elastic band 8x60 cm
1 REF	00602.100	Velcro elastic band 8x100 cm
1 REF	MA321	Instruction's Manual

# INSTALLATION AND TREATMENT START

- Unpack the equipment and accessories, check that they do not have damages caused by the transport. If there are damages (of you have doubts), do not use the unit and / or the accessories and refer to a qualified staff.
- Check that the disposable main voltage is conforming to the one required.
- Put the main switch on O (switched off). Insert the supplied main cable into the socket.
- Connect the unit to a main socket of 220-240V / 50-60 Hz.
- Switch on the equipment by the main switch on I (switched on), on the frontal panel the display of the three parameters: frequency, intensity, time, , switch on.
- Insert into the outputs OUT one or two transducers.
- By the keys of the “Freq” section, set the desired frequency; by the keys of the “Level” section set the desired intensity and by the keys of the “Timer” section the desired time.
- Connect one or two transducers on the part to treat. Hold the transducers with the supplied bands and be careful they do not hold too much to avoid any impediment to blood circulation.
- To end the treatment it is sufficient to switch off the equipment.
- Environmental conditions:

	<i>STORING</i>	<i>WORKING</i>
Environmental temperature	between -10 and 50 °C	between 10 and 40 °C
Relative damp	between 10 and 85%	between 30 and 75%
Atmospheric pressure	between 50 and 106 kPa	between 60kPa and 106 kPa

## MEANING OF GRAPHIC SYMBOLS ON THE UNIT

Conforming to the European Directive 93/42/CEE

Read carefully the Instruction's manual before to use the equipment (symbol IEC-ISO 348) (CEI EN 60601-1)

Protection against direct and not direct contacts: Type BF (symbol IEC 878-02-03) (CEI EN 60601-1)

II Class device (symbol IEC-ISO 417-5172)

**SN**

Serial Number (symbol 4.4 UNI EN 980)

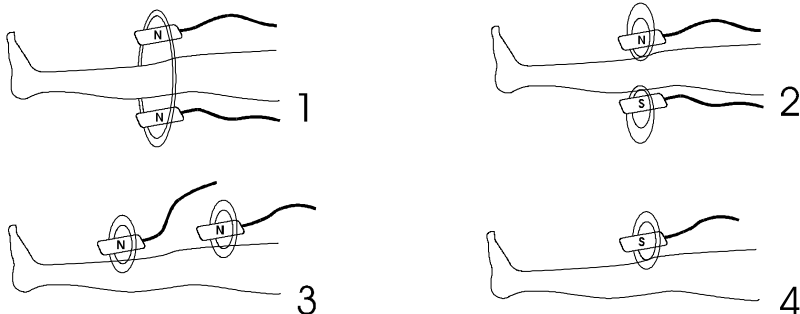
Manufacturing Date (symbol 4.5 UNI EN 980)

## PLACEMENT OF THE TRANSDUCERS

On the transducers is signed a side with the written N (North) and on the opposite side S (South).

Put the transducer / transducers in correspondence of the part or parts to treat, if it is necessary use the supplied elastic bands.

The different magnetic fields produced changing the transducers' positions are shown in the following figures. We have represented a leg, but obviously the connections are similar for other parts of the body.

1 – Use of the two transducers to produce a coherent concatenate field. A transducer in contact with the N part and the second with the S. The part to treat is between the two magnets. Produced field: **COHERENT**.

2 – Use of the two transducers to create a repelling field. The two transducers have the same polarity in contact with the part to treat. The part to treat is between the two magnets. Produced field: **REPELLING**

3 – Use of the two transducers for therapy of two parts or greter parts. the two transducers are put with the same part in contact. The part to treat is between the two magnets. Produced field: **NORMAL-WIDE**.

4 – Use of one transducer. The transducer is directly put on the part to treat. Produced field: **NORMAL**.

## APPLICATIONS

The information given in this chapter are explicative, they are not a therapeutic guide and do not replace the medical opinion. The treatment's modality that must be indicated by medical staff authorized legally. No responsibility is by the supplier of the unit, for the consequences of the treatments, without suitable check of medical staff.

### *INDICATIVE TABLE OF THE APPLICATIONS*

<i>Treatment</i>	<i>FREQUENCY (Hertz)</i>	<i>LEVEL (Gauss)</i>	<i>TIMER (minutes)</i>	<i>SESSION (number)</i>	<i>Position</i>
ABDOMINAL COLICS	50	45	60	5 - 10	WIDE (3)
ACUTE SINUSITIS	50	45	30	10-15	NORMAL (4)
ADYNAMIA SENILE	10	30	20	10-15	NORMAL (4)
AMPUT. PAIN	50	60	20	10-15	NORMAL (4)
APOPLEXY	10	45	60	10 - 20	NORMAL (4)
ARTERIAL DEASES	50	80	25	10-15	WIDE (3)
ARTERIOPATHY OBL.	10	60	30	8 - 12	NORMAL (4)
ARTHRITIS	99	45	60	10 - 15	COHERENT (1)
ARTIC. INFLAMM.	30	80	30	8-12	NORMAL (4)
ARTROSIS	25	60	60	10 - 15	COHERENT (1)
ASTHENIA	50	45	30	10 - 20	NORMAL (4)
BONY ULCER	10	20	10	10-15	COHERENT (1)
BRONCHIAL ASTHMA	10	30	30	10 - 15	COHERENT (1)
BRONCHITIS	25	45	30	10 - 15	COHERENT (1)
BURNINGS	10	80	60	10-15	COHERENT (1)
BURSITIS	25	80	60	8 - 10	NORMAL (4)
CELLULITIS	10	60	60	10 - 15	NORMAL (4)
CERVICAL ARTROSIS	10	30	10	8-12	WIDE (3)
CHRONIC SINUSITIS	10	45	60	10-15	NORMAL (4)
CONTUSIONS	30	30	15	8-12	NORMAL (4)
COXOARTROSIS	10	60	15	10-15	COHERENT (1)
CROHN 'S MORB	25	80	30	10-15	COHERENT (1)
DEGEN. MOT. APPAR.	50	60	20	10-15	COHERENT (1)
DERMATITIS	10	40	10	8-10	REPELLING (2)
DISCOPATHY	30	50	20	10-15	WIDE (3)
DISPEPSIE	10	80	30	8 - 12	WIDE (3)
DISTORSIONS	50	80	60	8 - 12	COHERENT (1)
DYSMENORRHEA	50	80	15	4-8	NORMAL (4)
EDEMA	10	20	10	8-12	NORMAL (4)
EPICONDILITIS	50	80	60	10 - 15	NORMAL (4)
FIX. ENDOPROSTESIS	25	60	10	10-15	COHERENT (1)
FRACTURES	25	80	25	10 - 15	COHERENT (1)
GASTRALGIA	50	45	60	8 - 12	WIDE (3)
GASTRITIS	50	80	25	10-15	WIDE (3)
GONOARTROSIS	50	80	30	10-15	REPELLING (2)
HEADACHE	99	15	10	10 - 15	NORMAL (4)
HEART INSUFFIC.	10	50	20	10-15	NORMAL (4)
HEMICRANIA	99	15	10	8 - 12	NORMAL (4)

<i>Treatment</i>	<i>FREQUENCY (Hertz)</i>	<i>LEVEL (Gauss)</i>	<i>TIMER (minutes)</i>	<i>SESSION (number)</i>	<i>Position</i>
HERNIA	10	80	60	4 - 8	NORMAL (4)
HYPERTONIA	10	50	20	10-15	NORMAL (4)
INFECTIONS	10	80	60	8 - 12	REPELLING (2)
INSOMNIA	50	30	60	10 - 15	COHERENT (1)
IPERTROPHY PROST.	50	60	30	10-15	NORMAL (4)
ISCHIALGIA	10	40	20	10-15	NORMAL (4)
LIMB NEUR.	99	60	30	10 - 15	NORMAL (4)
LUMBALGIA	50	60	60	10 - 15	NORMAL (4)
LUMBOSCIATALGIA	50	50	20	10-15	COHERENT (1)
LUXATIONS	10	80	60	10 - 15	COHERENT (1)
MIOSITI	60	80	40	8-10	NORMAL (4)
MOTORY APPARATUS	10	60	15	10-15	COHERENT (1)
MUSC. ATROPHY	10	80	60	10 - 15	COHERENT (1)
MUSCUL. PAIN	99	60	30	10 - 15	COHERENT (1)
MUSCUL. TEARS	25	80	30	8-12	COHERENT (1)
NEPHRITIS	50	30	60	10 - 15	NORMAL (4)
NEPHROSCLEROSISI	10	15	60	10 - 15	COHERENT (1)
NEPHROSIS	10	30	60	10 - 15	COHERENT (1)
NEUR. DISFUNCTION	99	15	20	5 - 10	WIDE (3)
OSTEOCHONDROSIS	10	40	15	10-15	COHERENT (1)
OSTEOPATHY	10	50	10	10-15	COHERENT (1)
OSTEOPOROSIS	10	80	60	10 - 15	COHERENT (1)
OVARITIS	50	80	10	10-15	NORMAL (4)
PARESIS	10	20	10	10-15	NORMAL (4)
PERIARTHRITIS SCAP.	10	50	10	10-15	COERENTE (1)
PHANTOM PAIN	10	30	60	10 - 20	NORMAL (4)
POLI ARTHRITIS	10	80	30	10-15	COHERENT (1)
PSEUDORTROSIS	10	80	60	10 - 15	COHERENT (1)
RHEUM. ILLNESS	99	30	60	10 -15	COHERENT (1)
SENILE HYPOT.	30	60	20	8-12	NORMAL (4)
SKIN ULCER	10	80	60	10-15	NORMAL (4)
SLEEP DEASES	30	40	10	8-12	NORMAL (4)
SORES	50	80	30	10-15	NORMAL (4)
SPASTIC PARESIS	50	80	30	8-12	NORMAL (4)
SPONDYL. DEFORM.	50	80	25	10-15	NORMAL (4)
SUDEK'S MORB	30	40	15	10-15	COHERENT (1)
TENDINITIS	25	60	30	8-12	COHERENT (1)
TONSILLITIS	50	80	60	8-12	NORMAL (4)
TRAUM. LESIONS	10	80	60	10 - 15	NORMAL (4)
TRIGEMINAL NEUR.	50	15	20	10 - 15	NORMAL (4)
ULCARATIVE COLITIS	25	80	30	10 - 15	NORMAL (4)
VERTEBR. PAIN	30	60	25	10-15	COHERENT (1)
WOUNDS	25	60	60	8 - 12	NORMAL (4)

---

# MAINTENANCE

---

## ***PREVENTIVE MAINTENANCE***

### **DETACH THE ELECTRIC CONNECTIONS BEFORE PERFORMING ANY OPERATION OF MAINTENANCE.**

The preventive maintenance consists mainly:

- in before every use control of the state of the unit and of the accessories to examine the integrity of all the cables, the isolations, the wraps etc. projected to avoid the access to powered parts.
- in the periodic cleaning of the units regularly effect for maintaining of the unit at the best of functionality conditions and appearance.
- in the periodic cleaning of the accessories.

## ***CLEANING OF THE UNIT***

Clean the external cabinet and the control panels with a damp cloth: no use of abrasive or solvent products it is recommended. The use of ethyl alcohol or of detergents containing high percentage of ethyl alcohol is not allowed on the control panels.

Don't dip the unit in liquids. In case of penetration of liquids do not use the unit and address to professionally qualified personnel.

## ***CLEANING OF THE TRANSDUCERS***

Clean the transducers with a damp cloth: no use of abrasive or solvent products it is recommended. Don't dip the unit in liquids. In case of penetration of liquids do not use the unit and address to professionally qualified personnel.

## ***CLEANING OF THE BANDS***

For the cleaning of the bands do not use warm water; in case of use of detergents rinse with carefully.

### **NOTE:**

The unit has to be periodically controlled (at least once a year) by qualified personnel to ascertain the electrical safety about the measurement of the leakage current

## ***CORRECTIVE MAINTENANCE***

If a malfunction of the equipment is found it is suggested to check first of all the control setting of the unit. The unit and its accessories should have checked by professionally qualified personnel in case that they are exposed to external mechanical solicitations, for instance after a serious fall, to liquid dripping, to strong overheating (for instance direct light of the sun, fire), or if their functionality seems to have altered or if some part of the unit's cabinet it has broken, lacking, or displaced or if some connector or cable show signs of deterioration.

**PROBLEMS SOLUTION**

These are some easy solutions to the problems with **Magneto Base +**.

**Problem****Solution**

The main switch is switched on but the unit doesn't work.	Inspect if the main cable is correctly inserted.
Unit is switch on but there is not magnete field's output	Check if the warning light of the output is lighted. Check if the transducers are correctly connected on the outputs OUT.
Yellow warning is not lighted	Disconnect the transducers: if after this the light will light-on please refer to the assistance service.
At the switching on the unit does not work correctly.	Switch off the apparatus, wait for at least 30 seconds and after this time try a new switch-on.

If the problems are not overcome, please refer to the qualified staff to control yours **MT BASE PLUS**. Do not open the unit for any reasons, this could be dangerous and decline any warranty.

---

## F.A.Q.

---

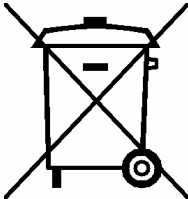
We report the **Frequently Asked Questions** about the Magnetotherapy.

	
<b>What is Magnetotherapy?</b>	Magnetotherapy is a kind of physiotherapy that uses, for a therapeutic intent, electromagnetic waves. It is a treatment legitimated by scientific studies and, developed to cure any disorders.
<b>How many types of magnetic fields are?</b>	There are constant magnetic fields and pulsed magnetic fields. Pulsed magnetic fields are divided in high frequency magnetic fields and in low frequency magnetic fields.
<b>What is a pulsed magnetic field?</b>	Pulsed magnetic field is a not alternated magnetic field (it do not invert its polarity), but pulsing.
<b>Why pulsed magnetic fields are used?</b>	Low frequency pulsed magnetic fields are an optimal therapeutic instrument for several pathologies, because they have cyclic course and not fixed. Thanks to this characteristic they are considered very effective
<b>How do you carry out the applications of Magnetotherapy ?</b>	The applications are carried out by supplied transducers that change electric impulse in magnetic field.
<b>How do Low Frequency and pulsed fields Magnetotherapy act?</b>	It recharges, regenerates the cells without force, influences the several processes that govern the functions of cellular membrane and has some effects at level of neuro-vegetative and cellular metabolic control.
<b>When a device for Magnetotherapy can be defined safe?</b>	To define a device for Magnetotherapy as safe, it must be according to the Standards in force. LED SpA is an Italian company, that projects and produces safe electro medical equipments that respect the Standards in force

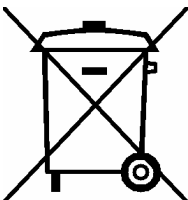
<b>Does Magnetotherapy have any contraindications?</b>	Magnetotherapy has not particular contraindications, but its use is advised on patients by pace-maker e, in pregnancy .
<b>Can I have results by using a device for Magnetotherapy?</b>	The effectiveness of the treatments of ELF and pulsed magnetic fields is legitimated by scientific studies as a solution for several pathologies.
<b>Where Magnetotherapy is applied?</b>	Traumatology, Reumatology, Angiology Neurology, Dermatology, Endocrinology
<b>How do you verify if you can use Magnetotherapy?</b>	It is recommended to consult a physician before to carry out the treatments.
<b>How long does a session of Magnetotherapy ?</b>	The session's time changes according to the person, the disease, the age, because the time of the cure depends from the disease and its entity.

*The information given in this chapter are explicative and they do not replace the medical opinion.*

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**Information about elimination of this product (Applicable in the European Union and other European countries with separate collection systems)**

	<p>On the end of the life, the present product <b>mustn't</b> be eliminated as urban refusal, but it must be eliminated in a separated collection.</p> <p>If the product is eliminated in unsuitable way, it is possible that some parts of the product (for example some accumulators) could be negative for the environment and for the human health.</p> <p>The symbol on the side (barred dustbin on wheel) denotes that the products mustn't throw into urban refuses container but it must be eliminated with separate collection.</p> <p>In case of abusive elimination of this product, could be foreseen sanctions.</p>
------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------